



FROM THE AGE OF 30, 50% OF WOMEN COMPLAIN ABOUT UNDER EYE CIRCLES¹

Stress, lifestyle, thin skin, hereditary factors...we are all unique when it comes to under eye circles

1. Arcane Research study conducted in France in February 2011 on 4,402 women aged 18 to 75.

Your eyes are a reflection of your personality

Your eyes express your emotions and feelings.

But when your eyes are shadowed by under eye circles, they lose their sparkle and shine.

These hollow depressions are caused by a loss of volume below the eyes in the area between the lower eyelid and the cheek, often accompanied by a darkening of skin colour.

Under eye circles can give the illusion of tired, dull and aged appearance. As a result, you don't feel like yourself anymore, and every day you try to conceal them.

Your medical practitioner can provide a long-lasting solution to treat under eye circles, using injection of a hyaluronic acid gel specifically developed by TEOXANE Laboratories for the delicate eye contour.



Hyaluronic acid: a medical solution to treat under eye circles

Hyaluronic acid is a natural component² of the skin. It holds in moisture like a sponge, to keep skin firm, soft and supple.

However, with age, the skin's hyaluronic acid content diminishes. The skin loses volume, becomes dry, thin and fragile. Underneath the skin, other tissues also retract.

As a result, under eye circles become hollow and more visible.

TEOXANE: Experts in hyaluronic acid

For over 10 years, TEOXANE Laboratories have specialised in designing hyaluronic acid-based injectable products. Our researchers have developed a biocompatible gel specifically designed for the delicate under eye area: TEOSYAL REDENSITY® 2.

2. Papakonstantinou *et al.* Hyaluronic acid: A key molecule in skin aging. *Dermatoendocrinol.* 2012; 4(3): 253-8.

TEOSYAL REDENSITY® 2 is a trademark of TEOXANE SA and it is a product of the range TEOSYAL® PURESENSE.


TEOSYAL® PureSense Redensity 2 is a medical device (class III) and is a regulated health product bearing the CE marking (CE0086) under this regulation. For more information, please consult your doctor or TEOSYAL® PureSense Redensity 2 instructions for use.



REDENSITY 2

An injectable gel specifically designed to treat under eye circles

A 3-in-1 patented formula^{*}

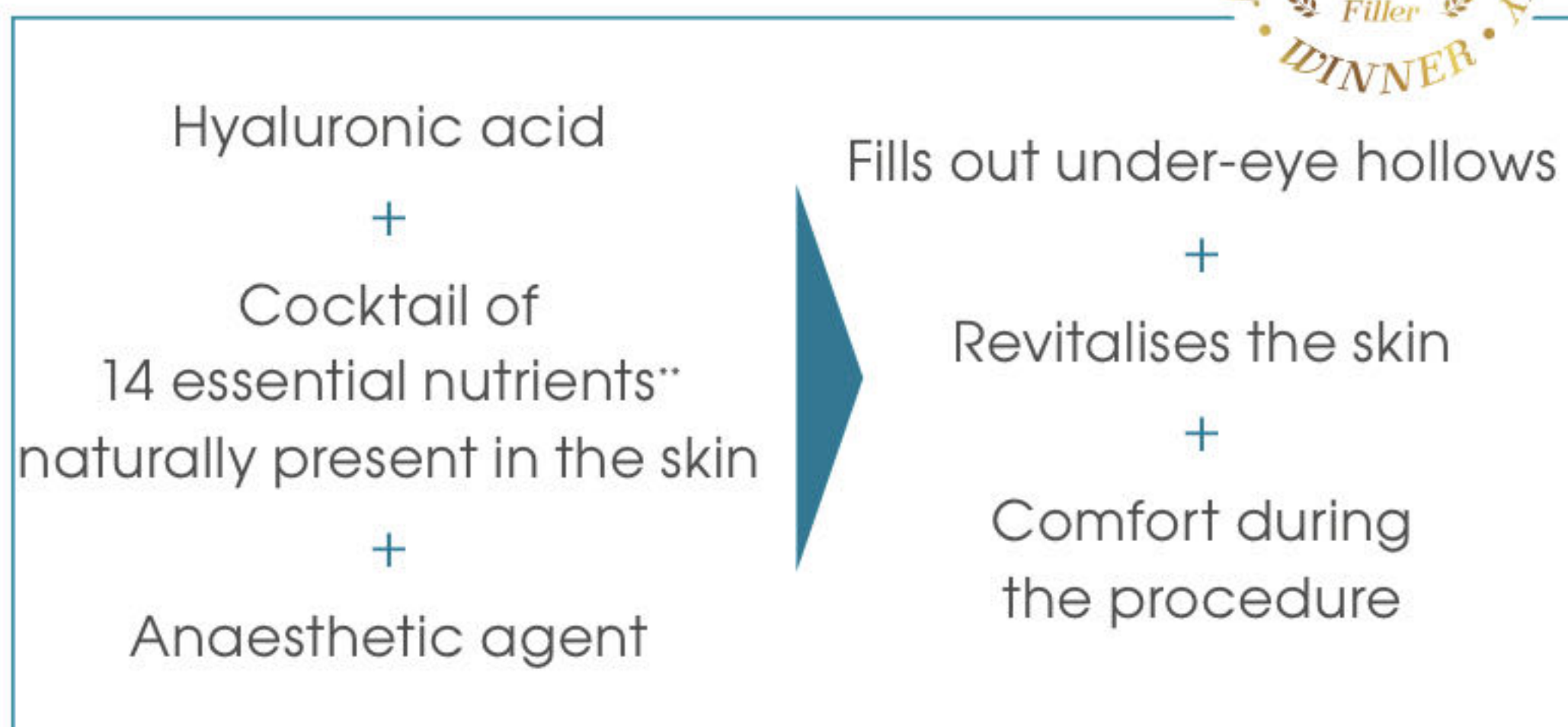

Immediate and long-lasting aesthetic results:

- 1• Under eye circle evenly corrected³⁻⁵
- 2• Hydrated and younger-looking skin³⁻⁵
- 3• Long-lasting global satisfaction, proven over 12 months⁶

^{*} FR1157848

^{**}8 amino acids (including 5 essential ones), 3 antioxidants, 2 minerals and 1 vitamin (B6). Essential amino acids cannot be made by the body. As a result, they must be supplied by the food or other supplements.

3. Multicentric and international study: 2012/2013. 33 practitioners, 151 patients. Indications: 93% of patients were injected in the tear trough area.

4. Berguiga M, Galatoire O. Tear trough rejuvenation: a safety evaluation of the treatment by a semi-crosslinked Hyaluronic Acid filler. *Orbit*, 2017;36 (1):22-26.

5. Park *et al.* Hyaluronic acid filler combined with antioxidants for infraorbital rejuvenation: Report of two cases. *Dermatologic Therapy*. (in press)

6. Multicentric clinical study of 60 patients followed during 1 year (2010). Several months after the injection, the hyaluronic acid is naturally eliminated by the body.

Frequently asked questions about under eye treatment using Redensity 2

What will happen during the treatment?

The medical practitioner will make a precise diagnosis of the area requiring treatment. An injection of the Redensity 2 gel will be administrated with a sterile, single-use syringe. The session will not take long, usually around 20 minutes.

How many treatments will I need?

Immediate results can usually be seen after just one visit. Occasionally, a touch-up injection may be needed within one month after your first session.

Can I go straight back to work after a session?

You may experience some post injection reactions such as minor bruising and/or swelling in the treated area. These reactions do not always occur and are usually mild, short term and correctable.

Ask your practitioner for advice and about possible side effects.*

TREATMENT INNOVATION

Dedicated to patient comfort during hyaluronic acid injection, TEOXANE Laboratories offer practitioners a new-generation injection system: the TEOSYAL®PEN.



An “injector pen” designed to:

- Reduce pain during treatment
- Reduce the intensity of local reactions (bruising, bleeding and swelling)

*There are potential side effects related to the implantation of this device, occurring immediately following injection or after delay. These include, among others : redness, oedema, papule. Patients should inform their practitioner of any secondary effects other than those described above or any effect persisting beyond one week.

3. Multicentric and international study: 2012/2013. 33 practitioners, 151 patients. Indications: 93% of patients were injected in the tear trough area.

4. Berguiga M, Galatoire O. Tear trough rejuvenation: a safety evaluation of the treatment by a semi-crosslinked Hyaluronic Acid filler. Orbit, 2017;36 (1):22-26.

TEOSYAL®PEN is a motorised and cordless device for hyaluronic acid injections to correct wrinkles and depressions. Manufactured by Juvaplus SA. This medical device (class IIa), is a regulated health product which bears, under this regulation, the CE mark (CE0434).

Don't put up with under eye circles.
Join the battle against them!

BEFORE



AFTER



© TEOXANE

IMMEDIATE OUTCOMES^{3, 4}

97% of patients were satisfied with the results

98% of patients
WOULD RECOMMEND THE TREATMENT^{3, 4}



To complete your Redensity 2
treatment at home:

R[II] EYES, an eye contour care formula containing hyaluronic acid. Fragrance-free and tinted, R[II] EYES offers an anti-fatigue and anti-ageing solution for the eye area.

Tested under dermatological control**

** In-vivo study carried out on 23 volunteers by an independent laboratory





In today's world, beauty is no longer a question of age. Women want their beauty to last into the future. That's why TEOXANE Laboratories specialise in the complex development of hyaluronic acid injectable gels with researchers driven by the culture of excellence and innovation as well as a total dedication to effectiveness and safety. Thanks to this approach, the hyaluronic acid gels we produce in our Geneva-based laboratories are of the highest quality and purity.

Mrs Valérie Taupin
CEO & Founder, TEOXANE

TEOXANE UK Ltd.

54-55 Shrivenham Hundred Business Park, Majors Road
Watchfield, Swindon Wiltshire SN6 8TY United Kingdom
www.teoxane.co.uk

TEOXANE Geneva

Les Charmilles - Rue de Lyon, 105 CH 1203 Geneva
www.teoxane.com



TEOXANE

THE EXCELLENCE OF SWISS SCIENCE